




VASO^{STAT}[®]

H E M O S T A S I S D E V I C E


REMOVAL GUIDELINES




Allow sufficient time to elapse to achieve hemostasis.
Typically 30-45 minutes.*

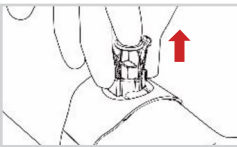


Squeeze wings and slowly withdraw plunger until it no longer contacts the skin. Verify hemostasis has occurred by visually inspecting the puncture site.

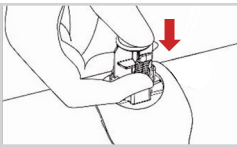

If bleeding, then reapply the plunger to stop the bleeding. Ensure pulse can be felt beyond the device. Check the device again in 10 to 15 minutes.

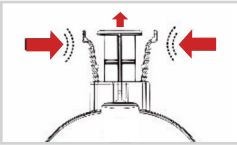

If no bleeding, then continue to slowly withdraw the plunger until it can be removed.

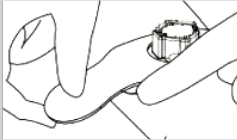

Peel off VasoStat and adhesive pad. Apply sterile dressing.

* Guidelines are based on compilation of best practices. VasoStat removal protocols should be consistent with needs of the patient and providers per facility protocol. Compression force and time to hemostasis may vary with each patient and procedure.

